

Warnings and Precautions

- Do not use RestoreX for more than 1 hour daily or for >30 minutes at a time.
- Separate 30-minute treatment sessions by at least 30 minutes.
- DO NOT wear RestoreX while sleeping
- DO NOT overtighten the self-adherent wrap or wear for more than 30 minutes.
- Discontinue use and consult a medical professional if:
 - You experience new or worsening pain in your scrotum, penis or abdomen
 - You develop a new curvature or indentation
 - You experience persistent skin discoloration
 - You experience pain with traction
 - You experience any worsening of sexual function
- Prior to each use, insure all screws adjusted during the set-up process are tight
- RestoreX is a single patient use device. Do not share with another individual.
- Never place RestoreX in a dishwasher or microwave oven.

Contraindications

Do not use RestoreX if:

- You have had penile, scrotal or pelvic surgery within the past six weeks unless otherwise indicated by your surgeon
- You experience chronic penile pain not related to Peyronie's disease
- You experience chronic testicular or scrotal pain
- You have open sores or wounds on your penis

RestoreX is not recommended for use in men with severe vascular disease including those who have:

- Absence of feeling in fingers, toes or limbs due to ischemia or neuropathy associated with diabetes or similar conditions, especially if the condition has resulted in loss of digits or limbs.
- Penile skin infections, open lesions or sores